


## Daily Check List

Read and understand the complete instructions in the user manual before operating the Advance

- ◇ Lift moves freely on its casters
- ◇ Cradle bar is free to rotate and swing
- ◇ Cradle bar is firmly attached
- ◇ Examine the sling hooks for wear
- ◇ Legs open and close correctly
- ◇ Hand pendant opens and closes the base
- ◇ Emergency stop button functions
- ◇ Slings are free of damage and are not fraying
- ◇ Battery is charged
- ◇ Fasteners are tight and do not show wear
- ◇ Emergency raise and lower buttons operate correctly
- ◇ Emergency descent operates correctly
- ◇ Wheel locks engage and disengage correctly for parking the lift

\* Electric version shown; hydraulic version also available

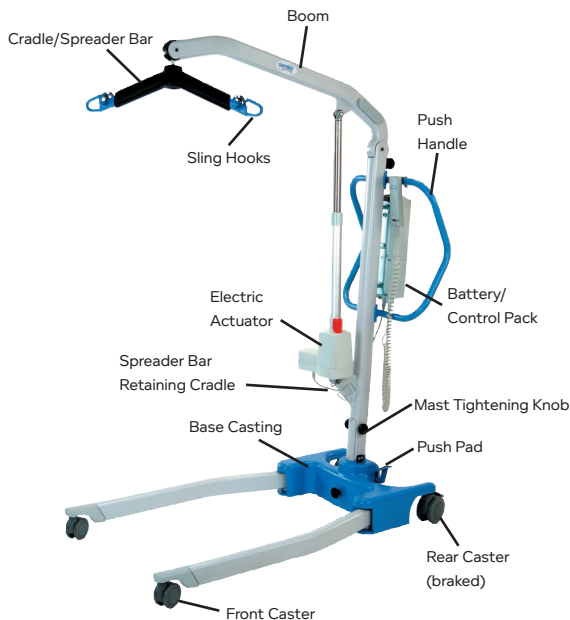


## Operating Instructions

1. Leg adjustment is achieved by pressing the buttons on the hand control.
2. The lift has two rear wheel braked casters. Brakes are to be used only for parking or storing the lifter.
3. Movement of the boom is achieved by pressing the up/down buttons on the hand control or by using the emergency lower/raise switches on the control box (see diagram).
4. The emergency stop button is located on the front of the control box.
5. The mechanical emergency release is a "Red" pull lever located on the lower portion of the actuator shaft.
6. The battery will sound an alarm when a charge is required. Batteries should be charged on a regular basis.
7. The 6-point cradle is standard on the Advance.

## Six Point Spreader Bar

Example of six point spreader bar sling with loop attachments.

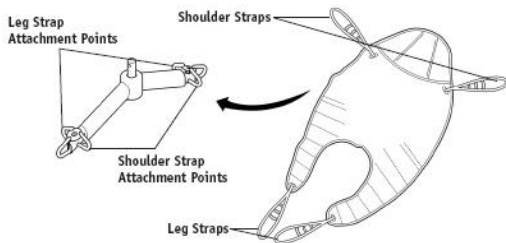

## Operating Controls

